



# ACROGEL

Marlic Medical Industries Co.  
No.597, Aboureyhan Blvd. Eshtehard Industrial  
City, Karaj - I R A N  
Tel : 0261 777 3201 Fax : 0261 777 3986  
Email : info@marlic.com Web site : www.marlic.com



**Alginate impression material, fast set – dust free  
Comply with ISO 1563/ADA 18 specification.**

---

## **INSTRUCTION FOR USE:**

- 1- Shake well the container before use.
- 2- Mixing ratio: 1 measure of powder (7 gr) to 1 measure of water (15 ml).
- 3- Mixing time: 35 sec.
- 4- Working time: 60 sec.
- 5- Time in the mouth: 60 sec.
- 6- Total setting time: 155 sec.
- 7- The temperature of water effect on setting time.
- 8- Above timing obtained with demineralized water at 23 c.
- 9- After taking impression, rinse it under cold running water, remove the excess water and pour immediately.
- 10- If you cannot pour immediately, store it in a tightly sealed plastic bag.
- 11- Make the plaster model within 48 hours of the impression being made.

## **Caution:**

- 1- Keep container in well ventilated place.
- 2- Store securely closed in a cool dry place.
- 3- Keep out of reach of children.
- 4- Do not empty to drains.

---

Our technical advise- wether verbal, in writing or way of trials – is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as their suitability for the intended processes and used. The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should in spite of this, liability be established for any damage, it will be limited to the value of goods delivered and used by you we will of course, provide consistent quality of our products within the scope of our general condition of sales and delivery.