



ACROPARS BF

Marlic Medical Industries Co.
No.597, Aboureyhan Blvd. Eshtehard Industrial
City, Karaj - IRAN
Tel : 0261 777 3201 Fax : 0261 777 3986
Email : Info@marlic.com Web site : www.marlic.com



ACROPARS BF – PLASTER COATING SOLUTION

ACROPARS BF / POWDER FORM

ACROPARS BF is made of best quality raw materials and its solution, forms a perfect barrier between plaster and acrylic, ensuring that clean, accurate mouldings are produced.

INSTRUCTION FOR USE:

- 1- In an opaque 1.5 lit bottle, pour 1 lot cold water, note the bottle should be clean.
- 2- Weigh 25 gr of **Acropars BF** and add to bottle. Mix them for 30 sec by spatula. And close the bottle.
- 3- For one week, mix the content of bottle every day for 30 sec. after that, the separating liquid is ready for use.
- 4- Spread the **Acropars BF** solution by the aid of a clean brass on the surface of plaster and let it to dry.
- 5- Take care; the artificial teeth don't be covered by solution.
- 6- Please store the **Acropars BF** solution in a cold place (i.e. refrigerator)
- 7- Please keep the **Acropars BF** powder in completely closed container and in dry places.