



ACROPARS OP

Marlic Medical Industries Co.
No.597, Aboureyhan Blvd. Eshtehard Industrial
City, Karaj - I R A N
Tel : 0261 777 3201 Fax : 0261 777 3986
Email : Info@marlic.com Web site : www.marlic.com



ACROPARS OP – Cold Cure Acrylic For Orthodontics Comply with ISO 1567

ACROPARS OP is a cold curing polymer for making orthodontics appliances, based on methacrylate and having very good mechanical properties. It can be used in dough or sprinkle on methods.

INSTRUCTION FOR USE:

Dough method:

- 1- seal the model with **Acropars BF** cold mold seal and place the wireframe on the cast.
- 2- Pours the liquid into mixing vessel and add powder to it (2.5 wt liquid, 5 wt powder). Mix then with a spatula for 10 sec.
- 3- After 1 minute from start of mixing, put the dough on the cast and press by your finger to shape it.
- 4- Put the cast into presser pot for 10 minutes then trim and polish the orthodontic appliance.

Spinkle method

- 1- Seal the model with **Acropars BF** cold mold seal and place the wireframe on the cast.
- 2- Moisthen the surface of cast by monomer then sprinkle powder on the cast to be absorbed into liquid, repeat above procedure till to achieve the desired thickness.
- 3- Polish the surface of appliance by liquid and put the cast into pressure pot for 10 minutes.

Note: Do not use **ACROPARS 200** powder and liquid with other brands.

- 1- Highly flammable.
- 2- Irritating to eyes and respiratory system, may cause sensitization by skin contact.
- 3- Keep container in well ventilated place.
- 4- Keep away from source of ignition.
- 5- Store securely closed in a cool dry place.
- 6- Keep out of reach of children.
- 7- Do not empty to drains.

Our technical advise- wether verbal, in writing or way of trials – is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as their suitability for the intended processes and used. The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should in spite of this, liability be established for any damage, it will be limited to the value of goods delivered and used by you we will of course, provide consistent quality of our products within the scope of our general condition of sales and delivery.