



ACROPARS Re

Marlic Medical Industries Co.
No.597, Aboureyhan Blvd. Eshtehard Industrial
City, Karaj - IRAN
Tel : 0261 777 3201 Fax : 0261 777 3986
Email : Info@marlic.com Web site : www.marlic.com



ACROPARS RE – Cold Cure Acrylic For Repair Comply with ISO 1567

ACROPARS RE is a cold curing polymer for dental prostheses based on methyl methacrylate and having very good mechanical properties.

Application:

ACROPARS RE (repair acrylic) is for repair , extending, and relining, of dentures and completing metal dentures.

INSTRUCTION FOR USE:

ACROPARS RE can be used in pourable and dough consistency.

Pourable consistency:

- 1- prepare the surface of denture which is broken or should be relined.
- 2- Mix 5 wt. of powder with 3.6 wt. of liquid in a mixing vessel by a steel spatula.
- 3- After 1 minute from start of mixing spread the paste on the surface of denture and put the denture into pressure vessel for 10 minutes.
- 4- Finish and polish the denture in standard techniques.

Dough consistency:

Dough method always is used for making full or partial dentures or extending dentures.

In this method, the procedure is same as Acropars 100 (heat cure acrylic for denture base). The ratio of components is 5 wt. of powder and 2.5 wt. of liquid.

Note: Do not use **ACROPARS Re** powder and liquid with other brands.

Caution:

- 1- Highly flammable.
- 2- Irritating to eyes and respiratory system, may cause sensitization by skin contact.
- 3- Keep container in well ventilated place.
- 4- Keep away from source of ignition.
- 5- Store securely closed in a cool dry place.
- 6- Keep out of reach of children.
- 7- Do not empty to drains.

Rev. No : 0 Doc. No: ES703

Our technical advise- wether verbal, in writing or way of trials – is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as their suitability for the intended processes and used. The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should in spite of this, liability be established for any damage, it will be limited to the value of goods delivered and used by you we will of course, provide consistent quality of our products within the scope of our general condition of sales and delivery.